





#### **5 YEAR WARRANTY**

Your TAAV Model T2008 is covered by a McGloins-Supertex Pty Ltd warranty for a period of 5 years from the date of purchase against defects in workmanship and materials. The warranty is additional to the conditions and warranties which are mandatory as implied by the Trade Practices Act 1974.

The warranty only covers the unit not working to its design and manufacturing specifications and only covers units used in Australia and New Zealand.

The warranty is only valid with the information filled out on the coupon below and a receipt of purchase – 5 year warranty from date of purchase.

The warranty does not cover negligent handling, tampering, or if used not in accordance with instructions.

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.

To obtain service under this warranty, product should be returned to McGloins-Supertex Pty Ltd. (customer to shoulder cost of freight), with a copy of the sales receipt as proof of purchase, with complete customer details provided (please see below):

Name:
Contact Number:
Address:
State: ——— Postcode:
Email:
Date when unit was purchased:
Place of purchase:
Reason for Return:
Please return a copy of this form to the below address when returning a faulty vaporizer:
McGloins-Supertex Pty Ltd.
PO Box 6847, Baulkham Hills BC, NSW 2153

Email: taav@mcgloins.com Web: www.mcgloins-supertex.com.au

7/7 Meridian Place, Bella Vista NSW 2153

Phone: 02 9836 1000 Fax: 02 9836 1055







# Vaporizer Vaporizer

# TAAV Model T2008 Instructions and Warranty



# BETTER THAN TRADITIONAL TWIST TOP VAPORIZERS

- Latest Technology
- Built in Auto Cut-Out
- Safety Night Light
- Ouietest Available
- Non-Stick, Base Mounted Element
- Produces More Steam for Longer
- Easy to Fill & Easy to Clean
- 12-14 Hours of Operation
- 4 Litre Capacity

IMPORTANT: FOR OPTIMUM OPERATION OF YOUR VAPORIZER, PLEASE READ THE INSTRUCTIONS BEFORE OPERATING THE MACHINE. FAILURE TO FOLLOW THE INSTRUCTIONS MAY VOID YOUR WARRANTY. ENSURE VAPORIZER IS RINSED AFTER EACH USE.

WARNING: KEEP OUT OF REACH OF CHILDREN. STEAM CAN BURN.











#### INSTRUCTIONS

 Remove unit from box and ensure the power cord is unraveled and any transit packaging removed. Failure to unravel the cord may cause overheating and possible fire hazard.

**CORD CAUTION:** Do not damage, forcibly bend, pull, distort or bind the power cord. Do not place heavy objects on it, or allow the power cord to be caught or pressed. If the power supply cord is damaged, it must be replaced by the manufacturer in order to avoid a hazard. Be sure to hold the electrical plug, not the cord, when disconnecting from the electrical source or the unit.

- 2. **Fill the unit while unplugged.** Always disconnect cord at main socket before filling or when not in use.
- Fill the unit and add inhalant. Pour water through the "grate" on top of the unit. Fill the unit only up to the water level as indicated. DO NOT OVERFILL.

Add 5–15ml of TAAV Mentholaire Vaporizer Inhalant directly into the water. The inhalant can be poured through the **triangular** grate found on the top of the unit.

**Do not use non-TAAV Inhalants**. The warranty may be voided if anything other than TAAV brand is used in the unit.


4. Place the Vaporizer on a stable position. The Vaporiser should always be on a firm, flat surface, 1.2 metres away from the patient. IMPORTANT: Vaporizer should be placed on the floor, where it cannot fall or be pulled down. Place on any surface that cannot be damaged by water.

CAUTION: Do not direct steam flow directly at patient, walls or furniture.

## KEEP OUT OF REACH OF CHILDREN; STEAM CAN CAUSE BURNS.

The appliance is not intended for use by young children or infirm persons without supervision. Young children should be supervised to ensure that they do not touch the appliance.

- 5. OPERATION: The unit will automatically cut off when the water has boiled dry OR if the heating element overheats. To reset the safety switch, unplug unit from the powerpoint, re-fill with water, wait for 10-15 minutes and then plug in.
- VENTILATE ROOM: Always leave a window or a door open when the unit is in operation. All steam-producing products can cause condensation and affect furnishings, paintwork, wallpaper and other fittings.

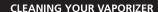

AFTER EACH USE: Empty any unused water from the Vaporizer

There will be residue particles which are created from minerals found in the water source left in the unit, rinse these out by half filling the vaporizer with warm water swirling around and emptying by turning over rocking from back to front in a pendulum motion over a sink or bath. Turn upside down and alow to dry before storing. The exterior of the unit can be wiped clean with a damp soapy cloth. **DO NOT IMMERSE IN WATER.** 

For outer metropolitan, hard water areas and regular users this rinsing should be done 2-3 times after each use.

**CONDITION OF SALE:** As a condition of sale, the purchaser assumes responsibility for the proper care and use of this appliance in accordance with our printed instructions. The purchaser or user must make his or her judgment as to when to use the appliance and the length of use.

### **IMPORTANT POINTS**

**DO NOT dismantle the unit.** Persons other than the manufacturer must not disassemble or modify the unit. **Modifying, forcibly opening or damaging the unit will void the warranty.** 

**Do not overfill the unit.** Overfilling can lead to water overflowing or spitting and cause damage to furnishings and flooring. The manufacturer will take no responsibility for damage caused by the unit due to misuse.

**Do not use non-TAAV inhalants in your vaporizer.** TAAV Mentholaire is specially formulated for your Model T2008 Vaporizer.

**Automatic cut-off:** The unit will automatically cut-off when the water has boiled dry or if the heating element overheats. To reset the safety switch, unplug unit from the powerpoint, re-fill with water, wait for 10-15 minutes and then plug in. This allows the unit to cool down and for the safety switch to reset.

**Safety Light:** The light on the ON/OFF switch will remain lit until the power is turned off at the powerpoint. Light will go off when water has boiled dry.

Keep clear of the steam outlet vent to avoid scalding.

Retain these instructions for future reference.

If you have questions regarding the functionality of the unit you have purchased or think that the unit you have purchased may be faulty, DO NOT DISMANTLE or MODIFY the unit.

Please ring McGloins-Supertex Customer Service on 02 9836 1000 during business hours, or visit our website: www.mcgloins-supertex.com.au for more information.

Please retain sales receipt with your proof of purchase for future reference.





